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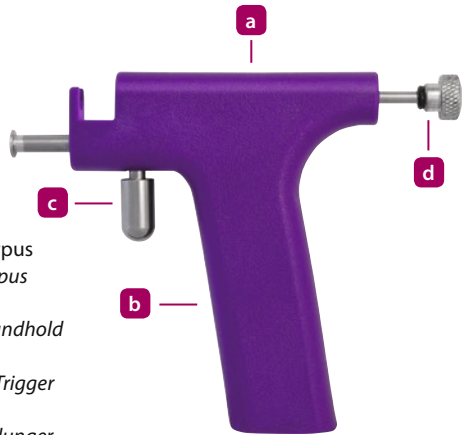
Operating Instruction Ear Piercing System

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container is gliding open-close without sticking.

7 Mark the ear with the pen provided.

8 Align the stud with the mark on the ear. Hold the instrument forward and the ear with your free hand.

9 Slowly pull the trigger until resistance is felt, then squeeze the trigger fully to fire the instrument.

10 When the instrument has fired release the trigger and lower the instrument from the ear towards the bottom.

11 To remove the used container pull back the plunger.

12 Remove the used container and dispose of hygienically.

GB

- 1 In the box you will find the EASYPIERCY classic ear piercing instrument.
- 2 This is what your instrument will look like when you take it out of the box.
- 3 Pull back the plunger.
- 4 Align the plastic container with the end of the instrument.
- 5 Push the plastic container onto the instrument until it is fully seated.
- 6 Pull back the trigger until the location pip is broken away. If necessary repeat this until the

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GB Before piercing, carefully wipe down the surface and customer chair used for piercing with the Disinfectant for Surfaces. Ask the customer to make him or herself comfortable in the chair.

Enter the customer's details on the Agreement Form and read the form out to the customer. Most importantly, inform the customer that there is a risk of infection following ear piercing as a result of inadequate after-care, metal intolerance or a number of other reasons, whereby the main cause is not carefully following the after-care instructions. External factors such as swimming or other sporting activities also increase the risk of infection.

Next, ask the customer to read through the form again and sign it if he or she agrees with the contents. Keep the signed section for at least 5 years, storing it, for example, in the EASYPIERCY file.

We would recommend only piercing the ears of customers aged 5 or older as the shape of the ear continues to change during the first few years.

Ask the customer to choose a starter stud. Ask where he or she would like his or her ear to be pierced and enter this on the agreement form as well.

Wash your hands and dry them with a disposable towel. Now disinfect your hands with the Disinfectant for Hands and put on the disposable gloves.

Ask the customer to remove any earrings that are already being worn. Ascertain whether there are any inflammations, raised skin or rashes on the ear. Also feel the relevant area between thumb and forefinger. If you detect any of the aforementioned abnormalities or any swelling, do not pierce! Ask the customer to see a doctor and to come back when these features are no longer visible.

Once you have ascertained that the ear can be pierced, mark the chosen place with the Marking Pen. Show your customer the mark with a

hand mirror and ask for confirmation.

Next, disinfect the relevant area. Leave the disinfectant to soak in for 30 seconds.

Now proceed as described in the operating instruction for the EASYPIERCY Classic ear piercing instrument on the front page.

If your customer would like to have a second ear piercing in the same ear, pierce this only after the first ear piercing is healed. The healing process usually takes 4 weeks. Tip: Pierce ear holes only in the area of the earlobe, as this part is better supplied with blood and thus more easily healed.

Between earlobe and the head of the ear stud and the earrings screw is a small distance. This supports the healing process including the needle tract. Apply EASYPIERCY ear piercing hygiene gel (item no. 500896) on the new ear holes in these places. Give your customer the remaining content for household use. Later, the ear studs and the earrings screws can be adjusted by the customer.

Ask the customer to remain seated for a few more seconds and explain how newly pierced ears should be cared for over the first few days. Hand the customer the After-Care Instructions for use at home. Tell your customer to see a doctor should the new piercings become inflamed or similar!

Wipe down the entire work surface and customer chair with the Disinfectant for Surfaces. Throw away the disposable towels used for this and then your disposable gloves too in the rubbish bag of the bin provided. Now disinfect your hands once more with the Disinfectant for the Hands.

Ask the customer whether he or she feels able to stand up. If not, for example in the event of dizziness, the customer should remain seated until he or she feels better.

EASYPIERCY is a trademark of Beco Technic GMBH

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Easy Piercy® ist eine Marke der | is a trademark of